

## MIDSTREAM PREGNANCY TEST

Pregnancy Test is a rapid one step that means hCG (human Chorionic Gonadotropin) in urine, a hormone that your body produces during pregnancy.

- \* Easy to read
- \* Results in 5 minutes
- \* Use any time of day
- \* Can test as early as the first day your period is late

### Performing the test

Note For in vitro diagnostic use only. Only for external use.

1. Remove the Pregnancy test device from the foil pouch.
2. Remove the cap and hold the absorbent tip directly in your urine stream for 5-10 seconds. If you prefer, you may urinate into a clean, dry cup or container and dip the absorbent tip in the urine for 5-10 seconds.
3. Lay the test device on a flat surface with the two windows facing upward. Results can be read within 5 minutes.

### Positive Test Results

If you see a line in each window, the test indicates that you are pregnant.

**Note:** One line can be darker than the other.

The two lines can be any shade of pink-purple and can be lighter or darker than the control. However, you should see two clear parallel lines.



### Negative Test Results

If you only see a line in the smaller Control Window and no line is present in the larger test Window, the test is negative indicating that you are not pregnant.

### Inconclusive:

The test is inconclusive if there is no line in the smaller Control Window. The test should be repeated using a new test kit.

