



PAP Link
BMC Medical Co.,Ltd.

PAP Link Quick Guide

1. Install PAP Link



For iOS users:

Search “PAP Link” in App Store



For Android users:

Search “PAP Link” in Google Play




2. Register your Account

Following these steps to get your own PAP Link Account:




Click “My Page”

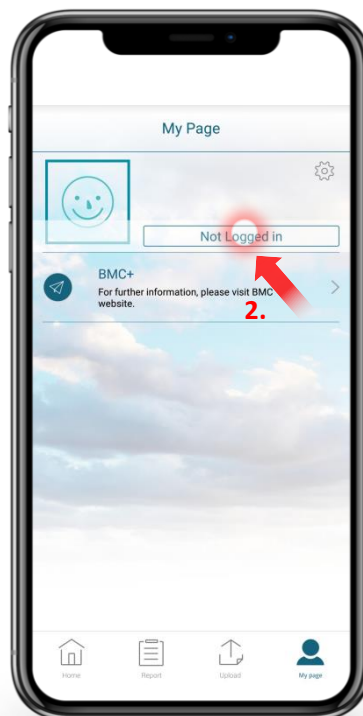


Click “Not Logged in”




PAP Link
BMC Medical Co.,Ltd.

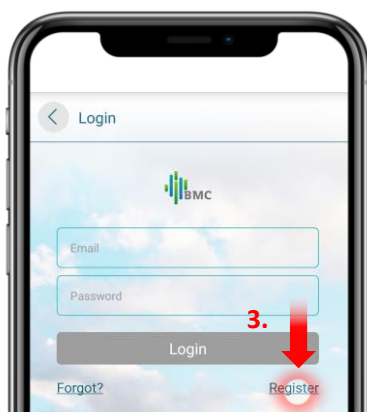


Click “Register” to get your Account

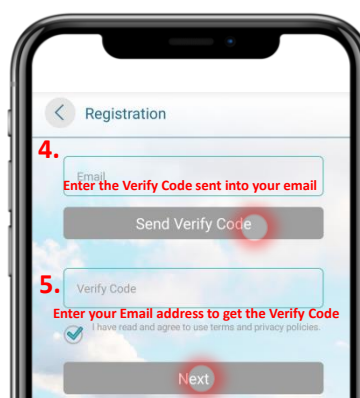


Your own Account would be
created after these steps

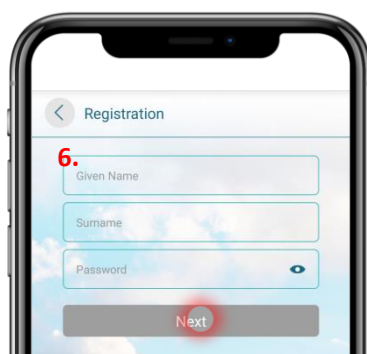


Set your name and password

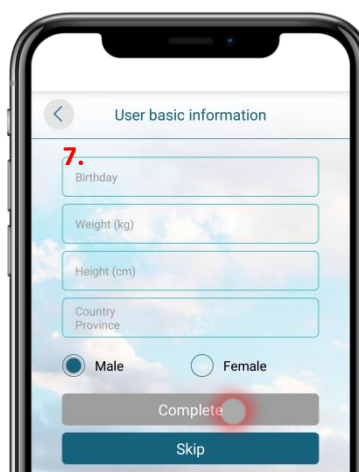

Fill out personal information or skip

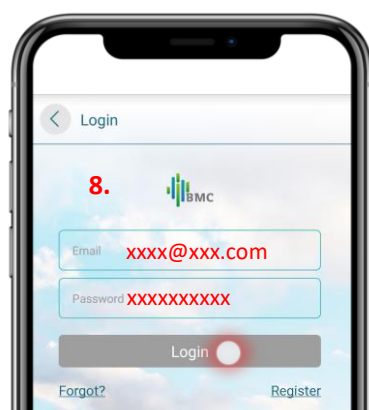

Enter your registered email
and password to login

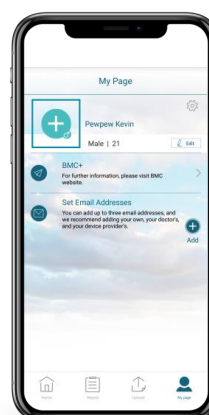

Complete all these steps to get your account
registered and logged in






PAP Link
BMC Medical Co.,Ltd.


3. Get your Report

With following steps to get your report:

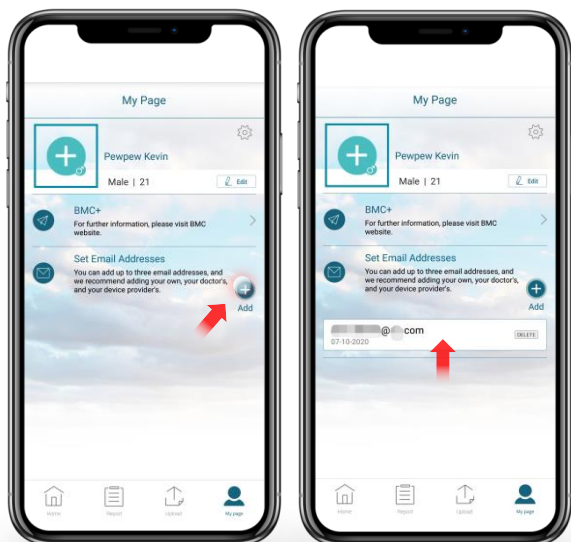


An account allowed to bind up to 3 email addresses.

Once you scan the QR code from devices, report would be automatically sent to the email.

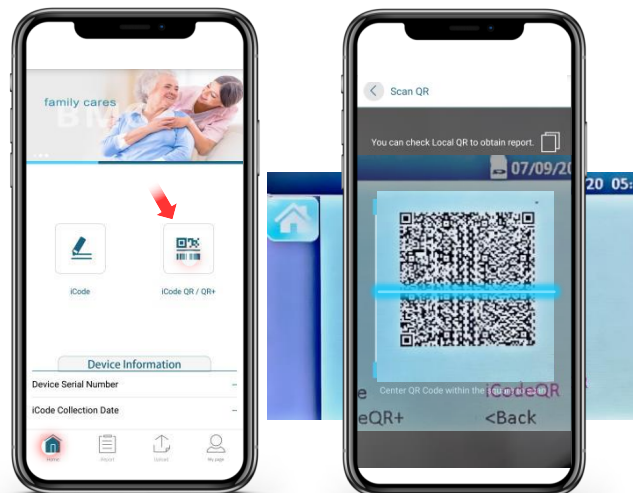

Device Information can be checked in the bottom of "Home" Page.

Click "iCode QR/QR+" or "iCode" to scan QR code from devices or enter iCode number to get your report.

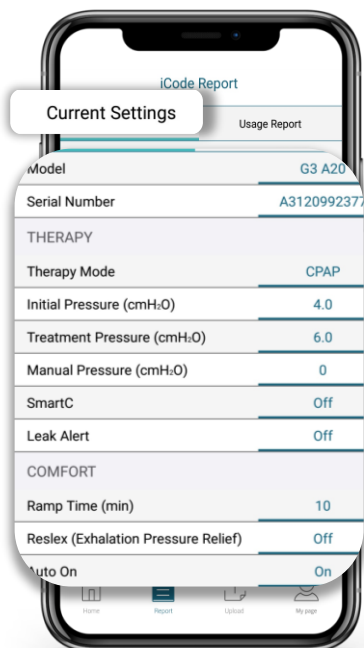


Current Settings of device would be showed for you.




Slide the screen for reports from different periods.




Tips for Changing Language:

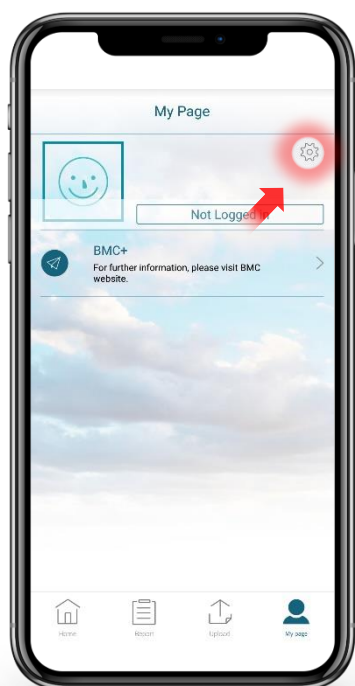

Click “Settings” icon.

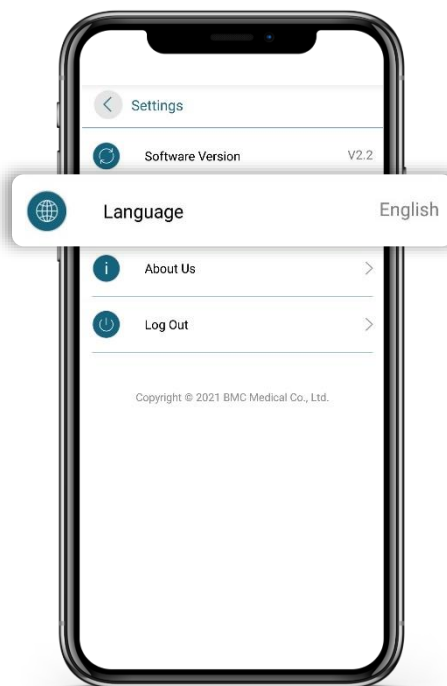

Selecting “Language”.

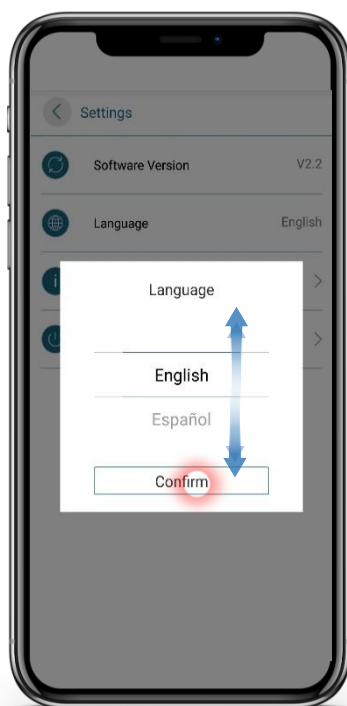

Slipping to choose targeted language and clicking “Confirm” to change language.

Demo Report:

Therapy Report

Report Date: Oct. 28, 2016

Patient Information

Name: Gender: - Age: - Height (cm): - Weight (kg): -
E-mail: @ com

Device Information

Model: G3 A20 Serial Number: A3120992377

Current Settings

Therapy

Therapy Mode: CPAP SmartC: Off
Initial P: 4.0 cmH2O Manual P: 0.0 cmH2O
Treat P: 6.0 cmH2O Leak Alert: Off

Comfort

Ramp Time: 10 min Humidifier: 2
Reslex: Off Heated Tubing: -
Auto On: On Mask Type: Full Face
Auto Off: Off Tubing Type: 22 mm

Statistics

	Last 1 day	Last 7 days	Last 30 days	Last 60 days	Last 90 days	Last 182 days	Last 365 days
Total Usage							
Used Days	1	1	1	1	1	1	1
% Used Days	100.0%	14.3%	3.3%	1.7%	1.1%	0.5%	0.3%
Used Days >= 4 hours	0	0	0	0	0	0	0
% Used Days >= 4 hours	0.0%	0.0%	0.0%	0.0%	0.0%	0.0%	0.0%
Used Time (hour:min)	02:54	02:54	02:54	02:54	02:54	02:54	02:54
Average Daily Used Time (hour:min)	02:54	02:54	02:54	02:54	02:54	02:54	02:54
Pressure							
Average Treat P (cmH2O)	10.0	10.0	10.0	10.0	10.0	10.0	10.0
P95 (cmH2O)	20.0	20.0	20.0	20.0	20.0	20.0	20.0
Respiratory Indices							
Apnea Hypopnea Index	23.4	23.4	23.4	23.4	23.4	23.4	23.4
Apnea Index	0	0	0	0	0	0	0
Hypopnea Index	23.4	23.4	23.4	23.4	23.4	23.4	23.4
Central Apnea Index	0	0	0	0	0	0	0

BMC Medical Co., Ltd.

BMCAres App - iCode QR

1/2

	Last 1 day	Last 7 days	Last 30 days	Last 60 days	Last 90 days	Last 182 days	Last 365 days
Oximetry							
Oxygen Desaturation Index	-	-	-	-	-	-	-
Average SpO2 (%)	-	-	-	-	-	-	-
Minimum SpO2 (%)	-	-	-	-	-	-	-
Average Pulse Rate (bpm)	-	-	-	-	-	-	-
Leak							
Average Leak (L/min)	0	0	0	0	0	0	0
95% Leak (L/min)	0	0	0	0	0	0	0
Large Leak (>90 L/min) Time (hour:min)	00:01	00:01	00:01	00:01	00:01	00:01	00:01
% Large Leak (>90 L/min) Time	1%	1%	1%	0%	0%	1%	1%